FACIAL TREATMENT CLEANING + PEELING + NUTRITION

HYDRA VACUUM

Indications: all skin types Aplication area: Facial

Lenght of treatment: 40 - 50 minutes

Frequency of the treatment:

Depending on skin type and professional opinion

SHOCK: 1 SESSION IN THE CABIN + DAILY CARE NORMAL: 4 SESSIONS IN CABIN + DAILY CARE INTENSIVE: 8 SESSIONS IN CABIN + DAILY CARE

BENEFITS:

- TIGHTENING EFFECT• DEEP CLEANSING
- IMPROVES SKIN RADIANCE AND APPEARANCE
- PORE REDUCTION WHITENING REGENERATION
- MOISTURISING

PRODUCTS:

- · BASIC LINE MILK
- · FACIAL CLEANSER CREAM BIJIN
- · BASIC LINE MICELLAR WATER
- · SALICYLIC ACID PREPEEL & POSTPEEL
- · HYDRA·VACUUM BOOSTERS
- · NATURAL ALGAE MASK + THERMAL WATER
- · DETOX SERUM

1. GETTING SKIN READY:

- 1.1: Clean skin with Basic Line Cleansing Milk, to remove makeup traces. Rinse with cold water
- 1.2: Apply a 3-4minutes massage with Facial Cleanser cream Bijin.
- 1.3: Rinse with cold water
- 1.4: Apply Basic Line Micelar Water
- 1.5: Apply SALICYLIC ACID.
 - Apply Prepeel with a gauze in face, neck and neckline.
 - After that apply SALICYLIC ACID with a fan brush. Leave it from 4 to 8 minutes.
 - After acid exposure remove excess of acid with a dry gauze and apply postpeel
 - Leave postpeel from 1 to 4 minutes
 - Rinse with water.

2. TREATMENT:

2.1: Select the most suitable enhancer according to the skin to be treated. Place in the machine.

2.2: Apply HYDRA VACUUM to the face, neck and décolleté (10 to 15 minutes) Use head 1 for general cleaning.

Use head 2 for areas with greater accumulation of dirt such as chin and brow.

Use head 3 for small areas such as the nose or localized comendones.

3. TREATMENT BOOSTER

NATURAL ALGAE MASK + THERMAL WATER

- 3.1 Place the matrix in a container and add 6 mL of activation liquid. The matrix is transformed from solid to liquid in a lubricating gel. Use a cosmetic brush to speed activation.
- 3.2 Spread the gel evenly over the skin, leaving out the hair line and the eyebrows.
- 3.3 Massage the skin for 10/15 minutes. The ingredients are released and used to work on the skin. Moisten again your fingertips, if necessary, to maintain the gel moist.
- 3.4 Remove any residue with a moist towel (warm or cold).

4. FINISHING:

4.1: Finish the treatment by applying **SERUM DETOX.**

5. DAILY CARE RITUAL:

- 5.1: Clean face, neck and décolleté with DEPURADORA BIJIN.
- 5.2: Recommend **DETOX KIT**

How to use:

During the seven days following the treatment:

At night: apply over the clean skin the content of one ampoule of salicylic acid and then In the morning: apply the serum.





